HD-GEL Electrode Gel

REF 15-0135

EN

HD-GEL Electrode Gel

Instructions for Use:

Description

HD-GEL Electrode Gel is a non-sterile, multipurpose, electrically conductive, water-soluble, non-staining, non-gritty electrode gel. It is bacteriostatic.

Indications for Use

HD-GEL is recommended for all electromedical procedures including ECG, defibrillation, biofeedback and EMG.

Warnings

- Do not use on patients with a known history of skin allergies to parabens.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

Directions for Use

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how HD-GEL should be used with their product. HD-GEL is water soluble, and can be removed using a moistened soft cloth.

Handling

No special handling required. HD-GEL is non-flammable and non-corrosive.

Storage

Protect product from direct sunlight.

Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.

EN	Instructions for use	KO	사용법
ES	Instrucciones de uso	ZH	使用说明
DE	Gebrauchsanweisung	AR	إرشادات الاستعمال
FR	Mode d'emploi	BG	Инструкции за употреба
IT	Istruzioni per l'uso	CZ	Návod k použití
PT	Instruções de utilização	ET	Kasutusjuhend
DA	Brugsanvisning	HR	Upute za upotrebu
FI	Käyttöohjeet	IS	Leiðbeiningar um notkun
NL	Gebruiksaanwijzing	LT	Naudojimo instrukcijos
SV	Bruksanvisning	LV	Lietošanas instrukcija
EL	Οδηγίες Χρήσης	NO	Bruksanvisning
HU	Használati útmutatás	RO	Instrucțiuni de utilizare
PO	Instrukcja stosowania	RU	Инструкция по применению
HE	הוראות שימוש	SK	Návod na použitie
JA	取扱説明書	SL	Navodila za uporabo

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